

MOLDEX[®]
Ideas that wear well.™

2200N Series

N95 PARTICULATE RESPIRATORS

2200N95 — Medium/Large

2201N95 — Small

2207N95 — Low Profile

Certified under 42CFR84.

Available in three sizes.

Dura-Mesh[®] shell resists collapsing in heat and humidity.


Molded nose bridge seals easily without a metal noseband.

Facepiece is naturally contoured without pressure points.

Straps are securely attached so they don't break at the seal.

Softspun[®] lining for increased comfort and durability.

Soft foam nose cushion for added comfort.

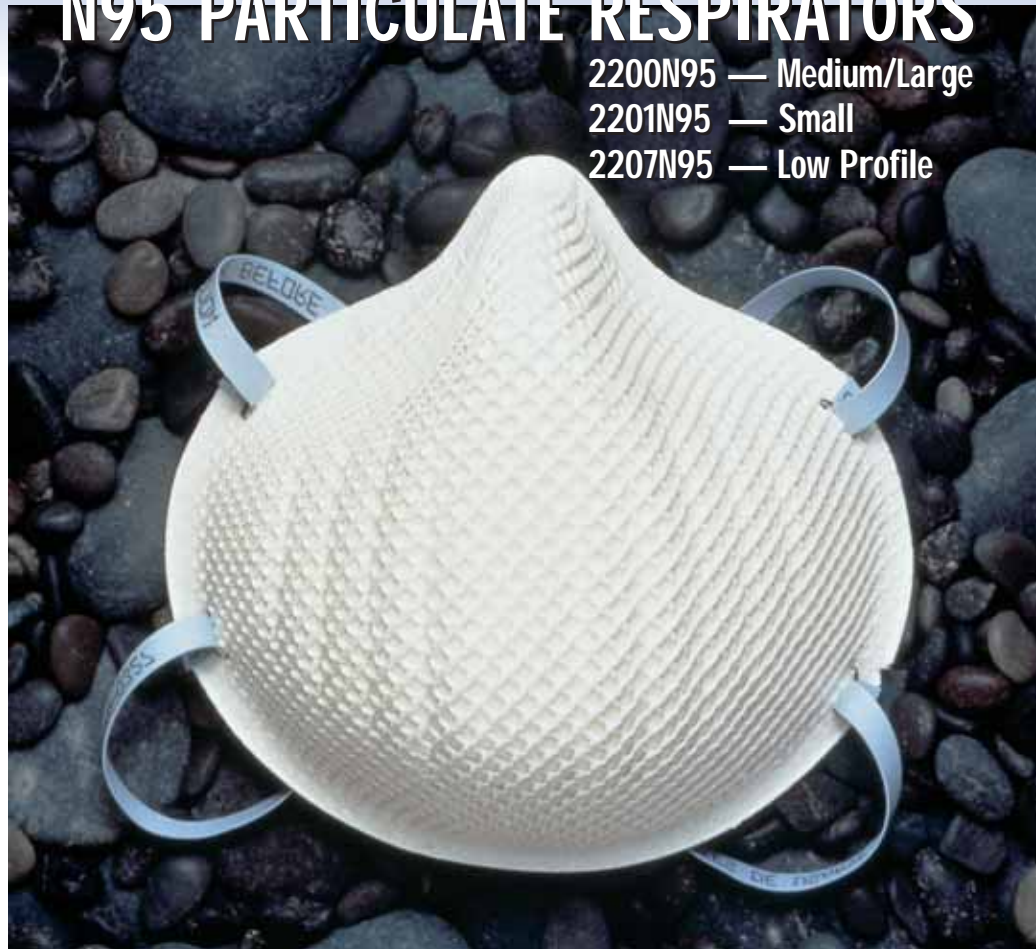

Medium/Large



Low Profile



Small

EASY FIT, MORE COMFORT.

The molded nose bridge with soft foam nose cushion provides the wearer an easy comfortable fit. There's no metal noseband to adjust. And the seal is maintained without pressure points.

LAST LONGER, USE FEWER.

The Dura-Mesh[®] shell, exclusive to Moldex, protects the filter media so it stays cleaner-looking longer. The shell resists collapsing, even in heat and humidity. So the respirator holds its shape longer. The result is fewer respirators used and lower costs.

FIT MORE WORKERS.

Available in three sizes to fit more workers and increase OSHA compliance. Use #2200N95 (M/L – blue straps) for most workers. For smaller faces #2201N95 (S – white straps) and #2207N95 (Low Profile – green strap) to fit Asian, African American and Native American workers.

MOLDEX TECHNICAL SUPPORT.

Moldex provides technical assistance including help with fit testing, a *Train-the-Trainer Program*, a manual on *How To Set-Up Your Respiratory Protection Program*, plus a Toll-Free Technical Help line. Just call 800/421-0668 ext. 550 or 310/837-6500 ext. 550.

**N95
PARTICULATE
RESPIRATORS**

2200N Series

2200N95 — Medium/Large
2201N95 — Small
2207N95 — Low Profile

APPLICATIONS

⚠ WARNING TO USER:

Follow all instructions and warnings on the use of this respirator and wear during all times of exposure. Failure to do so will reduce respirator effectiveness, wearer protection and may result in sickness or death.

Before use, the user must first be trained by the employer in proper respirator use, in accordance with applicable safety and health standards, for the contaminant and exposure level in the assigned work area.

The particulates and other contaminants which can be dangerous to your health include those which you cannot see, taste, or smell.

This product has not been sold with warnings or use instructions for personnel involved in healthcare or related situations, where there may be the possibility of contact with disease or biological hazards. If you are considering such uses, first call the Moldex Technical Service Dept., 310/837-6500 ext. 550 or 800/421-0668 ext. 550.

RESTRICTIONS:

1. This respirator does not supply oxygen and must not be used in atmospheres containing less than 19.5% oxygen.
2. Do not remain in contaminated area if any physical distress occurs, for example breathing difficulty, dizziness or nausea.
3. Leave contaminated area and replace respirator if it is damaged, distorted, a proper fit can not be obtained or breathing becomes difficult.
4. Do not alter, modify, or abuse this respirator.
5. Store unused respirators in box in a clean, dry, non-contaminated area.
6. Dispose of respirator according to your employer's policy and local regulations.
7. Dispose no later than thirty days after first use.
8. Use respirator before the "use by" expiration date printed on box.

USE AGAINST:

1. Particulate aerosols free of oil.

DO NOT USE AGAINST:


1. Concentrations of contaminants which are unknown or are immediately dangerous to life or health.
2. Concentrations of particulates which exceed the maximum use concentration or 10 times the OSHA Permissible Exposure Limit, whichever is lower.
3. Gases, vapors, asbestos, paint spray, sandblasting or particulate materials which generate harmful vapors.
4. Oil-based mists.

LIMITED WARRANTY


IMPORTANT NOTICE TO PURCHASER

This limited warranty is made in lieu of the warranties of merchantability, fitness for particular purposes and all other warranties, express or implied. There are no other warranties which extend beyond the description on the face hereof. The physical standards and specifications of Moldex will be met by products sold. **Exclusive Remedies:** damages for the breach of this limited warranty are limited to the replacement of such quantity of Moldex, products proved to be defectively manufactured. Except as provided above, Moldex shall not be liable or responsible for any loss, damage, or liability, direct, indirect, incidental, special, or consequential, arising out of sale, use, or misuse, or the inability to use products by the user.

NIOSH LABEL



MOLDEX/METRIC INCORPORATED
 CULVER CITY, CALIFORNIA, U.S.A.
 800/421-0668



THESE RESPIRATORS ARE APPROVED ONLY IN THE FOLLOWING CONFIGURATIONS

TC-	PROTECTION ¹	RESPIRATOR			CAUTIONS AND LIMITATIONS ²
		2200N95	2201N95	2207N95	
84A-0327	N95	X			ABCJMNOP
84A-2456	N95		X		ABCJMNOP
84A-2438	N95			X	ABCJMNOP

1. Protection

N95 Particulate Filter (95% filter efficiency level) effective against particulate aerosols free of oil; time use restrictions may apply

2. Cautions and Limitations

A- Not for use in atmospheres containing less than 19.5 percent oxygen.
 B- Not for use in atmospheres immediately dangerous to life or health.
 C- Do not exceed maximum use concentrations established by regulatory standards.
 J- Failure to properly use and maintain this product could result in injury or death.
 M- All approved respirators shall be selected, fitted, used, and maintained in accordance with MSHA, OSHA, and other applicable regulations.
 N- Never substitute, modify, add, or omit parts. Use only exact replacement parts in the configurations as specified by the manufacturer.
 O- Refer to user's instructions, and/or maintenance manuals for information on use and maintenance of these respirators.
 P- NIOSH does not evaluate respirators as surgical masks.

TECHNICAL HELP LINE

For information, technical assistance, and training materials call 800/421-0668 ext. 550 or 310/837-6500 ext. 550.

PACKAGING

20 Respirators per Box
 12 Boxes per Case
 240 Respirators per Case

WARNING

Failure to follow all instructions and warnings in the proper use of this respirator may reduce respirator effectiveness, wearer protection and may result in sickness or death. For additional information consult your supervisor, read all information on box and phone 800/421-0668 ext. 550 or 310/837-6500 ext. 550.

DISTRIBUTED BY:

MOLDEX-METRIC, INC.

10111 W. Jefferson Blvd.
 Culver City, CA 90232
 800/421-0668 310/837-6500
 FAX 310/837-9563

www.moldex.com e-mail: sales@moldex.com
 Moldex, Dura-Mesh and SoftSpun are registered trademarks.
 The Mesh Design is a trademark of Moldex.
 U.S. Patents #D285,374, #4,850,347, #4,384,577.
 Made in the U.S.A. 9700-768 REV G 9/05



In Canada:

Pacesetter Sales & Associates
 20 Arthur Hall Drive
 Sharon, ON L0G 1V0
 Tel: 905/478-8042 Fax: 905/478-8010
 www.pacesettersales.com